

AXIS AXRD-100 RAPID AUTO-READER



for incubation of rapid BI's



KORDON

Tıp Sağlık Araç ve Gereçleri

Manufacturer: KORDON TIP SAGLIK ARAC GERECLERI

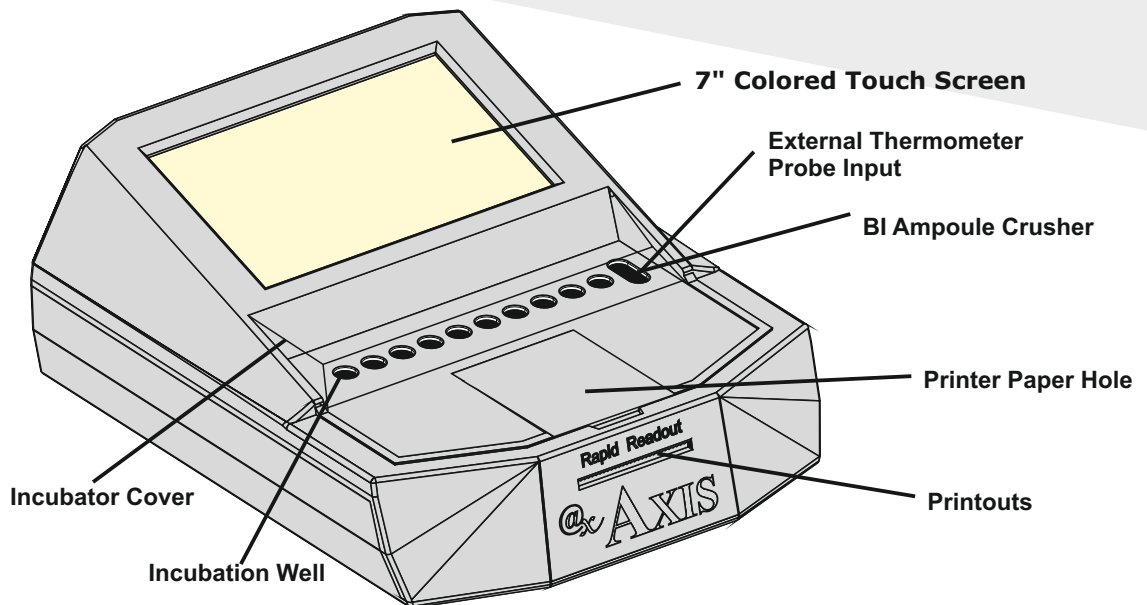
PROJ. MUH. ITH.SAN. VE TIC.LTD.STI.

10006/1 No:43 A.O.S.B CIGLI IZMIR TURKIYE


www.kordontip.com * info@kordontip.com

Tel: +90232.348.60.60 * Fax: +90232.348.60.90

Rapid Incubator; AXRD-100



Axis AXRD-100 Rapid Auto-Reader for incubation and readout of rapid biological indicators. 10 wells automatically detects the indicator and starts the incubation process. Allows quick, accurate and reliable detection of positive and negative results with no need for routine maintenance. Automatic system for incubation and reading of biological indicators for H₂O₂, Steam and Ethylene Oxide sterilization processes. Allows quick, precise, reliable and easy detection of positive and negative biological indicators, providing final results in short time with printer results. Allows selecting between two different incubation temperatures and running different incubation times simultaneously.

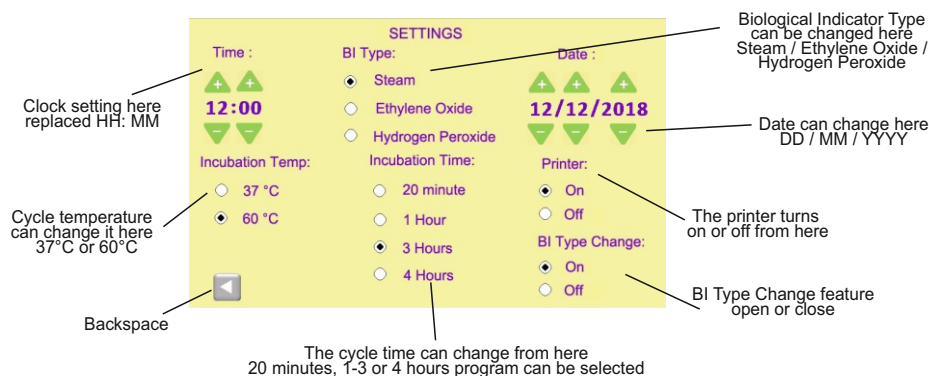

Axis AXRM-100 Auto-Reader user interface allows user to change incubation time 37 or 60°C, biological indicator type, incubation time, date and printer options.

- Compact size design, saving crowded bench space
- Multiple display and printing languages including English, Spanish, Portuguese, etc
- Automatic readout: Automatically detects the placed indicator and provides a readout
- Document sterilization results in a record log, using a thermal printer

-7" TFT LCD

```

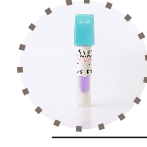
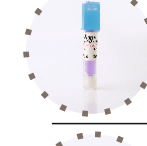
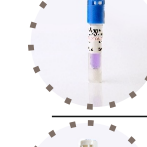
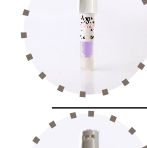
AXIS RAPID INCUBATOR S/N: A-017
Autoclave No:... Operator:...
PROGRAM : 3 HOUR / 60 C
BI TYPE : STEAM
DATE : 30/4/2019
START TIME : 15:10
READOUT TIME : 15:46
TEST TUBE NO : 7
RESULT : POSITIVE

AXIS RAPID INCUBATOR S/N: A-017
Autoclave No:... Operator:...
PROGRAM : 3 HOUR / 60 C
BI TYPE : STEAM
DATE : 30/4/2019
START TIME : 15:10
READOUT TIME : 15:46
TEST TUBE NO : 6
RESULT : POSITIVE

AXIS RAPID INCUBATOR S/N: A-017
Autoclave No:... Operator:...
PROGRAM : 3 HOUR / 60 C
BI TYPE : STEAM
DATE : 30/4/2019
START TIME : 15:10
READOUT TIME : 15:46
TEST TUBE NO : 2
RESULT : NEGATIVE
    
```



AXIS RAPID BIOLOGICAL INDICATOR REFERENCE CHART

Product Code	Temperature	Time	Medium Color	N (-)	P (+)
 AX110 EO	37°C	4 hr			
 AX220 Steam	60°C	3 hr			
 AX221 Steam	60°C	1 hr			
 AX222 Steam	60°C	1 hr			
 AX223 Steam	60°C	30 min			
 AX224 Steam	60°C	20 min			
 AX92 Plasma	60°C	1 hr			
 AX93 Plasma	60°C	20 min			

*A processed BI with a positive result (YELLOW) indicates a failure in the sterilization process.
USE AXRD-100 RAPID AUTO-READER TO INCUBATE RAPID BI'S.*