
Steam Chemical Integrator

Buhar Kimyasal Entegratör

@x **AXIS**[®]

TYPE 5

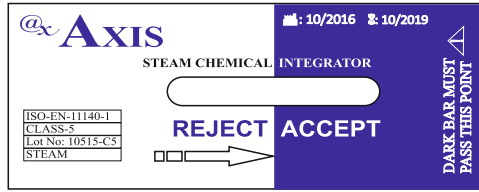Product Code/Ürün Kodu
AXCP15

Size/Ölçü: 60mmx30mm

500 pcs / adet

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Steam Chemical Integrator AXCP15

for monitoring Steam sterilization processes

Product description

Axis AXCP15 Steam Chemical Integrators are designed to react to Steam sterilization processes within loads, ensuring adequate control of the effectiveness of sterilization processes (temperature, time, and steam quality). It consists of a paper wick and a chemical pellet contained in a foil laminate, which melts and migrates as a dark bar along the paper wick through a zone marked ACCEPT or REJECT, thus indicating whether sterilization conditions were met.

Directions for use

1. Place the Axis AXCP15 Steam Chemical Integrators within each package, bag or tray that will be sterilized by Steam.
2. Make sure you have used enough amount of chemical integrators according to your country/hospital regulations and the size of the load.
3. Run the sterilizer as usual.
4. After the sterilization completed. Wait for at least 10 minutes to load cool down.
5. After processing, the dark bar should have reached the ACCEPT zone.
6. If the dark bar has not reached the ACCEPT zone, a REJECT result is indicated and the items in the pack, peel pouch, container system, or tray were not exposed to proper steam sterilization conditions.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.
Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

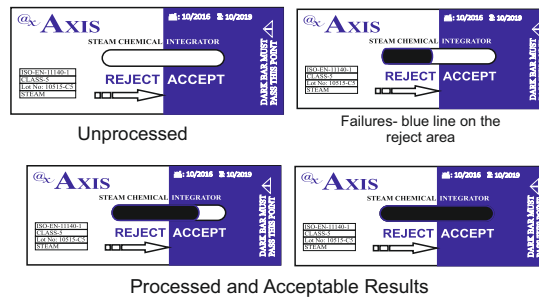
Important

If the indicator does not reached the ACCEPT zone, these may be the reasons;
The main failure causes are:

- Improper cycle selection
- Overloading / incorrect loading of the sterilizer
- Poor steam quality
- Autoclave malfunction

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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