
Steam Chemical Indicator

Buhar Kimyasal İndikatör

@x **AXIS**®

TYPE 6

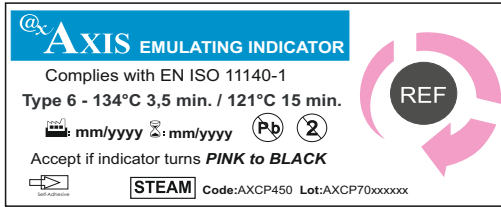Product Code/Ürün Kodu
AXCP450

Size/Ölçü: 60mmx30mm

250 pcs / adet

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Steam Chemical Indicator AXCP450

for monitoring Steam sterilization processes

Product description

Axis AXCP450 Adhesive Steam Chemical Indicators are designed to react to Steam sterilization processes within loads, ensuring adequate control of the effectiveness of sterilization processes (temperature, time, and steam quality). Pink indicator turns to reference color of Black when the process reaches the selected variables.

Directions for use

1. Place the Axis AXCP450 Adhesive Steam Chemical Indicators within each package, bag or tray that will be sterilized by Steam.
2. Make sure you have used enough amount of chemical indicators according to your country/hospital regulations and the size of the load.
3. Run the sterilizer as usual.
121 °C, 15 minutes or 134 °C, 3.5 minutes.
4. After the sterilization completed. Wait for at least 10 minutes to load cool down.
5. Compare the results from the given reference color chart. If the color change to reference color of black means that the established sterilization conditions were met.
6. After use, for recording, the indicator will remain its final color for 6 months. It is non-toxic, does not contain lead and does not transfers to materials to be sterilized. Axis AXCP450 Steam Chemical Indicators does not deteriorate or wrinkle during sterilization. **The back of the product is completely self-adhesive. It can be pasted into documentation documents for recording purposes.**

Storage

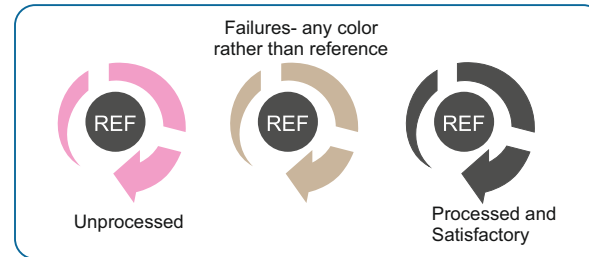
Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.
Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Important

If the indicator does not properly turn to Black color, these may be the reasons;
The main failure causes are:
• Improper cycle selection
• Overloading / incorrect loading of the sterilizer
• Poor steam quality
• Autoclave malfunction

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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