AXPCD222

with 1 hour AX222 Steam Biological Indicator & AXCP15 Steam Integrator
1 saatlik buhar biyolojik indikatör & AXCP15 Buhar Entegratör

AXIS

Quality Certification/ Kalite Sertifikası

AX222

Geobacillus stearothermophilus (ATCC 7953)

LOT

Population : CFU/KOB
Popülasyon

D-Value : min/dk
D-Değeri
(121°C)

Survival Time : min/dk
Hayatta Kalma Süresi
(Survival time = (log₁₀ labeled population - 2) x labeled D-value)
(121°C)

Kill Time : min/dk
Ölüm Zamanı
(Kill time = (log₁₀ labeled population + 4) x labeled D-value)
(121°C)

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AXCP15

LOT

SV (Stated Values)
121 °C.....
134 °C.....

PCD With Chemical Indicator

For monitoring of Steam sterilization processes

Product Description

Axis Process Challenge Device (PCD) Test Pack has been designed to simulate a load to be sterilized and to pose a challenge to the sterilization process. It is used to evaluate the effective performance of the process by detecting inadequate air removal and steam penetration. Furthermore, it allows to perform the routine monitoring and periodic validation of the sterilizers (after repair, installation, relocation). Each pack consists of a stack of porous cards holding a self-contained biological indicator tube that contains a population of *Geobacillus stearothermophilus* ATCC 7953 spores soaked on a carrier as well as growth indicator medium contained in a glass ampoule. The moving front chemical integrator shows ACCEPT result when sterilization conditions were reached while process indicator on PCD box changes from dark color when exposed to steam.

Instructions for Use

- 1.Place the pack inside a normally loaded steam autoclave, in those areas which are considered most inaccessible for the sterilizing agent (the center of the load and areas near the door).
- 2.Run the sterilization cycle.
- 3.After the sterilization process has finished, open the sterilizer door, wait for 5 minutes and remove the test pack. NOTE: The color of the box may vary from the original after undergoing the sterilization cycle. This does not represent a problem regarding the operation or quality of the product.
- 4.Check that the process indicator printed on box has changed color to dark color. Open the test pack, wait 5 minutes. Allow it to cool down to room temperature.
- 5.Check AXCP 15 moving front chemical integrator for correct exposure. If the dark bar has reached the ACCEPT zone, this confirms that the inside of the pack has been exposed to correct sterilization conditions.
- 6.Check the process indicator: A color change to brown confirms that the biological indicator has been exposed to steam. IMPORTANT: This color change does not indicate that the process was sufficient to achieve sterility.
- 7.Identify the AXIS AXPCD222 Fill out the required information on the Record Card.
- 8.Press the cap to seal the tube. Crush the glass ampoule contained in the AXPCD 222 with an individual ampoule crusher or with the ampoule crusher placed within the incubators incubation area.
- 9.Incubate the processed biological indicator and the indicator used as positive control in AXIS AXRD -100 Rapid Incubators. A positive fluorescence result (or culture medium color change after (incubation) means failure of sterilization process. If a negative fluorescence result is obtained (or culture medium remains the original color after incubation), it means that sterilization process was successful.
- 10.Record the AXPCD 222 and integrator results and adhere the self-adhesive Record Card or, alternatively, only the area containing the integrator indicator.
- 11.Discard the pack and the indicators immediately.

Precautions

WARNING: Do not use PCD for monitoring Ethylene Oxide, Dry Heat, Formaldehyde or any sterilization process other than Steam. Do not reuse biological indicators.
WARNING: Place one or more PCDs in sterilizing hard-to-reach areas to ensure all areas of the chamber are sterilized. Evaluate all load configurations to ensure ALL hard-to-reach areas have been identified, and place a PCD in each of those locations.
WARNING: Do not reuse the sterilizer until the biological indicator test result is negative.

Storage

Store in a dark place, at temperatures between 10-30 °C, 30-80 % relative humidity. Do not freeze. Do not store near sterilizing agents or other chemical products.

Shelf Life

AXPCD 222 has an expiration date of 2 years from the date of manufacture, when stored at recommended conditions. Do not use after expiration date. Chemical Integrators and Process Indicators have an expiration date of 2 years when used in/on PCD.

Disposal

Discard biological indicators after use according to your country's healthcare and safety regulations.

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MUH.PROJE.ITH.SAN.VE TIC.LTD.STI