Instructions for use:

1. Connect the AXIS AXRM100 rapid incubator to a power supply network for voltage set temperature and incubation time will appeared for 3sec on the right bottom of the screen.
2. The device will be started in 37°C or 60°C with a factory set incubation time .
3. The set temperature will reached in while. If the set temperature is higher then incubator temperature will see "Heating" and likewise "Cooling" sign appeared on the right bottom if the set temperature is lower .
4. When the AXRM100 Mini Auto Readout Rapid Incubator started the screen will be like (Figure 1).Continue by selecting the relevant cycle time, temperature and indicator type.

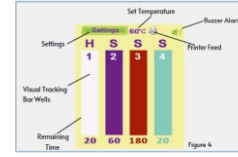
Figure 1

DIRECTIONS FOR USE:

5. Break the biological indicator, place it in a slot and wait for the result. Reading will start automatically. Once the biological indicator is placed in the slot, the progress bar for that slot will start counting down from the rented incubation time.

Readout Details

- EO / AX110 - 240min at 37°C
- STEAM / AX220 - 180 min at 60°C
- STEAM / AX221 - 60 min at 60°C
- STEAM / AX222 - 60 min at 60°C
- STEAM / AX223 - 30 min at 60°C
- STEAM / AX224 - 20 min at 60°C
- PLASMA / AX92 - 60 min at 60°C
- PLASMA / AX93 - 20 min at 60°C

Do not touch BI's during incubation!

6. At the end of the incubation period, the incubator gives a "POSITIVE" or "NEGATIVE" result for each biological indicator. These results can be output. Incubate the treated biological indicator and the untreated indicator for a positive control for the time indicated in it. Press the cap to turn off the biological indicator and break the ampule, make sure the media holder is thoroughly wet! A positive result indicates that a sterilization process error may have occurred. Identify the cause of positive BI by following existing facility policies and procedures. The Settings screen will appear when the user enters the password. From this screen, the user can edit the time, date, incubation temperature, incubation time and printer status.

- Steam and Hydrogen Peroxide readings can be taken simultaneously. For this, BI Type Change must be selected as ON in the settings section. Since Ethylene Oxide incubates at 37°C, the 4-hour Ethylene Oxide program should be done alone. The color of the incubation bars for the tracking visible on the main screen changes when you select the biomarker type and cycle time in the settings section. There are different bar colors for each BI and cycle time. Thus, it is aimed to minimize user errors such as wrong program or biological indicator selection.

passwords

- 1071** : SETTINGS Menu
- 8000** : LANGUAGE Settings
- 1453** : RECORDS
- 1111** : OPERATOR Information Setting

TROUBLE SHOOTING

- A. Temperature**
If the temperature does not appear on the display when the units is switched on, switch the device off and then on again. If still not appears unplug and plug the adapter while the switch is open. If the problem persists, contact your authorized service.
- B. Well Cancelled**
that has been canceled due to:
-BI removed and not placed in its well during the allowed time.
-Calibration problems.
-The user started incubation while there is a BI in the well.
- C. No Paper**
In this malfunction, the printer paper is out or loaded incorrectly. In this case, please check if there is paper in the printer and its correct placement.
- D. High/Low Temperature**
This mostly happens when the user switches from 60°C to 37°C to high temperature, and from 37°C to 60°C to low temperature.

- E. Put Back BI To Well**
During incubation, if the authorized/unauthorized user removes BI from it is will the incubator will give alarm. Since the removal of BI will affect the result it should be placed in well as soon as possible or should not be removed.

- F. Alarm**
If the user wants to stop audial alarm, press the Buzzer icon once to stop

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MUH. PROJE İTH. SAN. VE TİC. LTD. STİ
AXIS IS A BRAND OF KORDON TIP LTD.