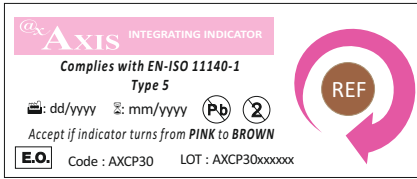
EO Chemical Indicator EO Kimyasal İndikatör

@x **AXIS**®

TYPE 5

Product Code/ Ürün Kodu
AXCP30

250 pcs / adet

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EO Chemical Indicator AXCP30 for monitoring Ethylene Oxide sterilization processes

Product description

Axis AXCP30 Ethylene Oxide Chemical Indicators are designed to react to Ethylene Oxide sterilization processes within loads, ensuring adequate control of the effectiveness of sterilization processes (temperature, time, relative humidity and gas concentration). Pink indicator turns to reference color of Brown when the process reaches the selected variables.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.

Do not put direct sunlight.

Do not re-use.

Do not store close to sterilizing agents.

Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Directions for use

1. Place the Axis AXCP30 Ethylene Oxide Chemical Indicators within each package, bag or tray that will be sterilized by Ethylene Oxide. Recommended placement: two CIs in opposite corners of each level.
2. Make sure you have used enough amount of chemical indicators according to your country/hospital regulations and the size of the load.
3. Run the sterilizer as usual.
For 30 min at 30°C or 20 min at 54°C. 60% RH and 600 mg/l EO.
4. It is recommended to use Axis AX10 Ethylene Oxide Biological Indicator for sterilization assurance.
5. After the sterilization and aeration completed. Wear protective cloths and remove Axis AXCP30 Ethylene Oxide Chemical Indicator from the load.
6. Compare the results from reference color. If the color change to reference color of Brown consider that the established sterilization conditions were met.
7. After use, for recording, the indicator will remain its final color for 6 months. It is non-toxic and does not contain lead. It does not deteriorate, wrinkles and transfers to materials to be sterilized.

Important

If the indicator does not properly turn to Brown color, these may be the reasons;

- Inadequate cycle duration
- Overloading / incorrect loading of the sterilizer
- Improper humidity
- Inadequate sterilization concentration

Disposal

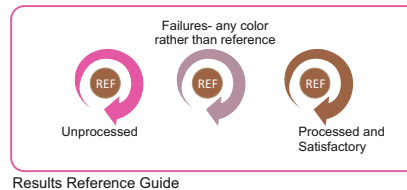
Discard after use, according to your country's healthcare and safety regulations.

Precautions

Do not store the product near sterilizing agent.

Do not expose this product to Dry Heat, Steam or any sterilization process other than EO.

Do not reuse the sterilizer until the indicator has changed into the color indicated in the product

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