Ethylene Oxide Gas Dosimeter

Etilen Oksit Gaz Dozimetre

@x **AXIS**[®]

Product Code/Ürün Kodu
AXCP380

50 pcs / adet

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Ethylene Oxide Gas Dosimeter

for monitoring EO sterilization processes

Product description

Axis AXCP380 is consist of plastic wrap and orange bar which reacts with Ethylene Oxide. Axis AXCP380 Ethylene Oxide Dosimeter checks important parameters while ensuring that the concentration can be reliably achieved. (sterile gas concentration, gas exposure time, temperature parameters, EO concentration, penetration, relative humidity) The orange bar on the indicator turn blue in the proportion to the dose of sterilizing gas, thereby showing whether the necessary conditions for sterilization are met. In order for sterilization to be considered successful, the color change from orange to blue must reach the area marked with at least an arrow. Thus, it indicates that it is understood that a dose of intake of 750 mg-hour/liter takes place. The triangular mark (▲) is placed on the dosimeter to indicate 750 mg-hour/liter EO exposure. EO concentration measurement is a requirement for parametric release of load.

Direction for use

1. Place the Axis AXCP380 Ethylene Oxide Gas Dosimeter with loads that will be sterilized by Ethylene Oxide in the sterilizer.

2. Run the sterilizer as usual.

3. After sterilization is completed, wait for 10 minutes to load cool down.

4. After processing, the dark zone bar have reached ▲ zone ▲ or above.

5. If the dark bar does not reached the ▲ zone, failure result is indicated and the items in the sterilization cycle were not exposed to proper ethylene oxide sterilization conditions.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.

Do not put direct sunlight.

Do not re-use.

Do not store close to sterilizing agents.

Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Important

If the indicator does not reached the ▲ zone, these may be the reasons;

The main failure causes are:

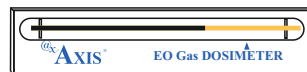
- Improper cycle selection
- Overloading / incorrect loading of the sterilizer
- Poor gase quality
- Autoclave malfunction
- Non-penetration

Disposal

Discard after use, according to your country's healthcare and safety regulations.

Unprocessed

Failure- dark bar does not reached ▲ zone

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