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Steam Autoclave Tape AXO19 & AXO25 for monitoring Steam sterilization processes

Product description

Axis Steam Autoclave Tapes with self-adhesive paper tape printed with indicator ink have been designed to monitor Steam sterilization processes, ensuring adequate exposure to the sterilizing agent during the sterilization process and allowing to distinguish between processed and unprocessed items.

Directions for use

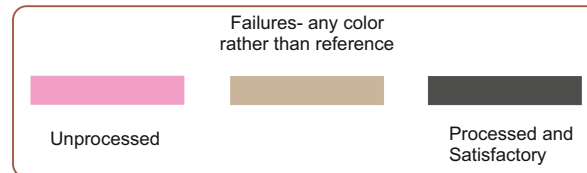
1. Place a wrapped package closed with Axis Steam Autoclave Tape that will be sterilized by Steam inside a sterilizer .
2. Axis Steam Autoclave Tape has 2 size;
AXO19 - 19 mm x 50 m
AXO25- 25 mm x 50 m
3. Run the sterilizer as usual.
3.5 min. at 134 °C / 15min. at 121 °C.
4. After the sterilization process has finished, remove the package from the autoclave and analyze the indicator results. The indicator must turn to the reference color for when it was exposed to Steam.
5. If a non-uniform or no color change is noticed, it should be reported immediately to the supervisor.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity. Do not put direct sunlight. Do not re-use. Do not store close to sterilizing agents. Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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