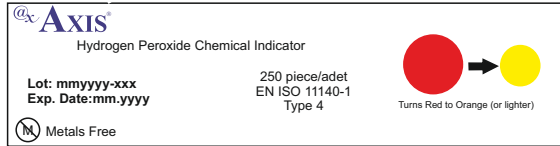
Hydrogen Peroxide Chemical Indicator Hidrojen Perosit Kimyasal İndikatör

@x **AXIS**[®]

TYPE 5

Product Code/ Ürün Kodu
AX40

250 pcs / adet

Size/ Ölçü : 20 mm x 80 mm

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Plasma Chemical Indicator For monitoring Hydrogen Peroxide sterilization

Product Description

Axis AX40 Hydrogen Peroxide Indicators are designed to react to Plasma sterilization processes within loads, ensuring adequate control of the effectiveness of sterilization processes. Red indicator turns to reference color of yellow when the process reaches the selected variables.

Directions for use

1. Place the Axis AX40 HP Chemical Indicators within each package, bag or tray that will be sterilized by Hydrogen Peroxide.
2. Make sure you have used enough amount of chemical indicators according to your country/hospital regulations and the size of the load.
3. Run the sterilizer as usual.
50 °C, 6 minutes, 2.3 mg/l H2O2.
4. After the sterilization completed. Wait for at least 10 minutes to load cool down.
5. Compare the results from the given reference color chart. If the color change to reference color of yellow means that the established sterilization conditions were met.
6. After use, for recording, the indicator will remain its final color for 6 months. It is non-toxic, does not contain lead and does not transfers to materials to be sterilized. Axis AX40 HP Chemical Indicators does not deteriorate or wrinkle during sterilization.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.

Do not re-use.

Do not store close to sterilizing agents.

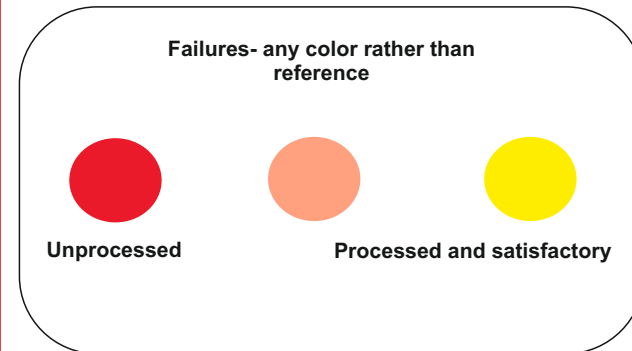
Shelf-life

Chemical Indicators have an expiration date of 5 years from the date of manufacture when stored at recommended storage conditions. Both the expiration date and batch number of the product are specified on the package label and in each unit of the product.

Do not use indicators after their expiration date.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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