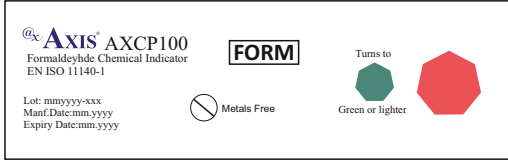
Formaldehyde Chemical Indicator

Formaldehit Kimyasal İndikatör

@x **AXIS**®

TYPE 4

Size/Ölçü: 60mmx30mm

250 pcs / adet

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Formaldehyde Chemical Indicator

for monitoring Formaldehyde sterilization processes

Product description

Axis AXCP100 Formaldehyde Chemical Indicators are designed to react to Formaldehyde sterilization processes within loads, ensuring adequate control of the effectiveness of sterilization processes. Red indicator turns to reference color of green when the process reaches the selected variables.

Directions for use

1. Place the Axis AXCP100 Formaldehyde Chemical Indicators within each package, bag or tray that will be sterilized by Formaldehyde
2. Make sure you have used enough amount of chemical indicators according to your country/hospital regulations and the size of the load.
3. Run the sterilizer as usual.
4. After the sterilization completed. Wait for at least 10 minutes to load cool down.
5. Compare the results from the given reference color chart. If the color change to reference color of green means that the established sterilization conditions were met.
6. After use, for recording, the indicator will remain its final color for 6 months. It is non-toxic, does not contain lead and does not transfers to materials to be sterilized. Axis AXCP100 Formaldehyde Chemical Indicators does not deteriorate or wrinkle during sterilization.

Storage

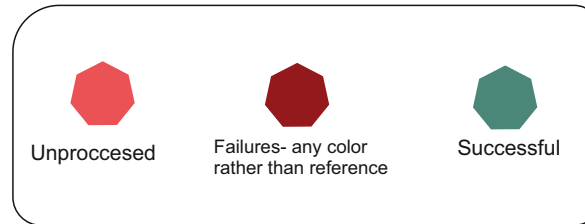
Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.

Shelf-life

Chemical Indicators have an expiration date of 5 years from the date of manufacture when stored at recommended storage conditions. Both the expiration date and batch number of the product are specified on the package label and in each unit of the product.
Do not use indicators after their expiration date.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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