# Bowie Dick Test Packs Bowie Dick Test Paketleri

AXIS®

Product Code/Ürün Kodu  
AXCP 140


Type 2  
200 sheet/ boxed  
200 yaprak/ kutulu

## Result Reference Guide/ Sonuç Referans Rehberi

UNPROCESSED/ İŞLEM GÖRMEMİŞ

UNSATISFACTORY/ YETERSİZ

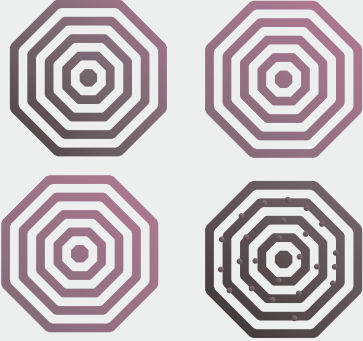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# Bowie Dick Test Pack for monitoring steam sterilization processess

## Product description

AXIS Bowie-Dick Test Pack is used to evaluate the effectiveness of air removal during pre-vacuum steam sterilization cycles operating at 134 °C, 3.5 minutes / 121 °C, 15 minutes. Pink indicator turns to reference color of Black when the load exposed to Steam sterilant. Test pack helps to determine sterilization malfunctions at early stage. A chemical indicator located on center on the package with porous materials in bottom and top .

## Instruction for use

1. Each day, before the first processed load, the Bowie-Dick test must be performed in an empty sterilizer to detect the presence of residual air in the chamber. Bowie-Dick Test is performed after a warm-up cycle at the beginning of every load.
2. Place Bowie-Dick Test horizontally on bottom shelf of sterilizer rack, over drain. Do not place directly on the drain or sterilizer
3. Run the sterilizer at 134°C - 3.5 min // 121°C-15 min
4. Remove the test from the sterilizer and wait until it cools.
5. A uniform color change to black or its shades indicates "SATISFACTORY". Non- uniform color changes indicates "UNSATISFACTORY". (Please see chart)
6. "SATISFACTORY" result indicates successful sterilization  
"UNSATISFACTORY" result indicates malfunction in sterilization process.

## Common causes of errors

Low pressure in the chamber  
Steam supply changes  
Leakages  
Poor steam quality  
Insufficient vacuum

## Caution

Hot surface. Please be careful to avoid skinburns before opening the pack. Do not store the product near sterilizing agents. Do not expose this product to EO, Dry Heat or any sterilization process other than Steam. Do not use a Bowie& Dick Test Pack in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase to check) or any kind of sterilization processes other than vacuum-assisted Steam. Do not reuse the sterilizer until the chemical indicator has changed homogeneously into the color indicated in the product.

## Disposal

Dispose according to the regulations of your institution and country.

## Shelflife

Chemical Indicators have an expiration date of 5 years from the date of manufacture when stored at recommended storage conditions. Both, product expiration date and batch number are specified on product packaging and in each individual card. Do not use indicators after their expiration date.

## Storage

Store in a dry place, keep away from sunlight, Temperature: 10-30 °C, relative humidity: 30-80 %.  
Do not store near sterilizing agents.  
It is recommended to protect the test sheets after use inside a plastic foil for storage.

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