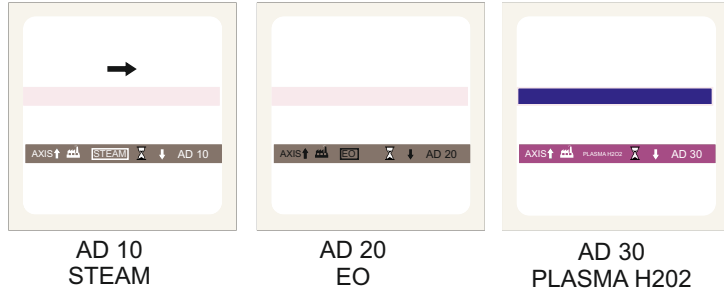
DOCUMENTATION LABEL DOKÜMANTASYON ETİKETİ

@x **AXIS**[®]
Product Code/Ürün Kodu
AXCP300

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DOCUMENTATION LABEL for monitoring sterilization processes

Product Description

AXIS Documentation Labels are chemical indicators for track and control sterilization processes. On the labels there is an ink printed chemical indicator. It provides minimum risk of errors, wasy and quick packaging and labeling. Labels can be used in Steam, Plazma and EO Gas sterilization processes allowing all packs to be labeled prior to sterilization.

Direction for use

1. To load the label roll
Open the baseplate by sliding the two plastic grips, one on each side of the body of the machine. The baseplate will fall open automatically. Open the label roll holder by pulling it upwards slightly. Unroll approx a 20 cm strip of label roll and insert it into the top of the machine, guiding the strip gently until it comes out of the bottom. Insert label roll into the roll holder and close it.

2. To insert the label strip
Before closing the baseplate, make sure to leave free at least 20 cm of the label roll. Insert loose end of the label roll into the slot behind the roller and push it slightly. Then, pull the handle until you see that the labels is correctly printed.

3. Print setting.
Move the arrow on the display to the required position by pulling the setting knob gently. Turn the setting knob out gently. Turn the setting knob until the required figure for that column appears on the display. Return the setting knob to its original position by pushing it towards the machine.

4. Label location adjustment and print setting
Loosen the screw on the guide plate to place the label correctly. Loosen the screws on the print head and then move it forwards of backwards as needed to make the print location higher or lower on the labels. Tighten screws.

5. To change in roller
Open the ink roller holder by pushing it outwards. Replace the ink roller by pressing gently as indicated by the narrow.

Storage

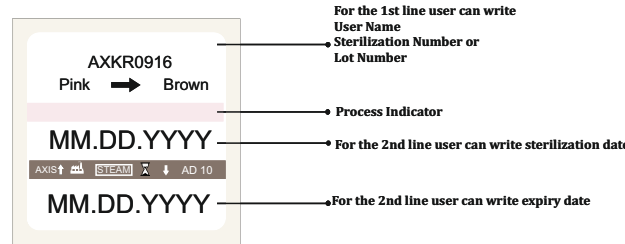
Store it in a dark place between 10-30°C, 30-80% relative humidity. Do not put direct sunlight. Do not store close to sterilizing agents.

Shelf Life

Chemical Indicators have an expiration date of 5 years from the date of manufacture when stored at recommended storage conditions. Both the expiration date and batch number of the product are specified on the package label and in each unit of the product.

Transport conditions

Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

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