Autoclave Tape Otoklav Bandı

AXIS®

Product Code/Ürün Kodu
AXE12/AXE19/AXE25

Type 2

Result Reference Guide/Sonuç Referans Rehberi

UNPROCESSED/ İŞLEM GÖRMEMİŞ

FAILED/HATALI

PASS/BAŞARILI

Manufacturer: Kordon Tıp Sağlık Arac ve Gereçleri Ltd.
10006/1 No:43 AOSB Cıgılı İZMİR TURKEY
Tel:+90 232 348 60 60
Fax: +90 232 348 60 90
www.kordontip.com - info@kordontip.com

Autoclave Tape

for monitoring Ethylene oxide sterilization processes

Product description

Axis EO Autoclave Tapes with self-adhesive paper tape printed with indicator ink have been designed to monitor EO sterilization processes, ensuring adequate exposure to the sterilizing agent during the sterilization process and allowing to distinguish between processed and unprocessed items. The adhesive component of the tape allows the adhesion to different types of packaging and wraps, such as cloth, paper and plastic being easy to remove and leaving no residues. Innovative highly sensitive cream ink is able to turn to dark brown / black when the indicator is exposed to Steam.

Directions for use

1. Place a wrapped package closed with Axis EO Autoclave Tape.
2. Axis Steam Autoclave Tape has 3 size;
AXE12-12mm x 50 m
AXE19 - 19 mm x 50 m
AXE25- 25 mm x 50 m
3. Run the sterilizer as usual.
(54 °C, 60 % RH, 600 mg/l EO)
4. After the sterilization process has finished, remove the package from the autoclave and analyze the indicator results. The tape indicator must turn to the reference color for when it was exposed to Steam.
5. If a non-uniform or no color change is noticed, it should be reported immediately to the supervisor.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.
Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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