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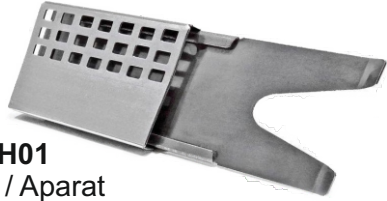
Cleaning Efficiency Indicator For Washers

AXCP120

Product description

Axis AXCP120 Cleaning Efficiency Indicators are designed for validation and routine monitoring of cleaning processes effectiveness in automatic machines including ultrasonic washing machines with different washing configurations. Axis AXCP120 Cleaning Efficiency Indicators allow a quick visual evaluation of the process and can identify potential sources of error.

Directions for use

1. Take a **Axis AXCP120 Cleaning Efficiency Indicator** and placed it on the **AXIS AX-WIH01 Holder**. Holder has been designed to mimics the real challenge of thoroughly cleaning hard-to-reach areas for the washing disinfecting process.
2. Place the **AXIS AX-WIH01 Holder** along with **Axis AXCP120 Cleaning Efficiency Indicator** inside and other material packages into the washer. Holder has hooks to ease placement on the tray.
3. Run the washer as usual.
4. Once the cleaning cycle is over, remove the holder with an indicator. Washing efficacy is influenced by detergent, water temperature, time and washing penetration. A good cleaning result is achieved by an appropriate combination of these variables.
5. Observe and record results according to the given color conversion chart. If unfavorable results do occur, users must check the quality/quantity of detergent, cleaning time, temperature as well as the general condition of the washer.
6. **Axis AXCP120 Cleaning Efficiency Indicator** has a writable structure that allows users to write notes on the indicator and use this label for recording/documentation.

Storage

Store it in a dark place between 10-30°C and 30-80 % relative humidity. Do not freeze and do not put direct sunlight. (this may deteriorate its coloring)

Important

Axis AXCP120 Cleaning Efficiency Indicator is a single-use product. Do not touch the indicator ink with your hands. **AXIS AX-WIH01 Holder** is reusable up to 100 cycles.

Disposal

Discard after use according to your country's healthcare and safety regulations.

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