Hydrogen Peroxide Tape Hidrojen Peroksit Bandı

@x
AXIS

**PRODUCT CODE/ÜRÜN KODU
AXH19**

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Hydrogen Peroxide Tape ¾" (19 mm x 50 m) AXH19

for monitoring Hydrogen Peroxide sterilization process

Product description

Axis Hydrogen Peroxide Tape with self-adhesive paper tape printed with indicator ink have been designed to monitor Hydrogen Peroxide sterilization processes, ensuring adequate exposure to the sterilizing agent during the sterilization process and allowing to distinguish between processed and unprocessed items.

Directions for use

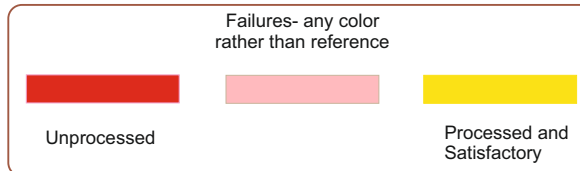
1. Place a wrapped package closed with Axis Hydrogen Peroxide Tape that will be sterilized by Hydrogen Peroxide inside a sterilizer.
2. An Axis Hydrogen Peroxide Tape has the size of 19 mm x 50 m
3. Run the sterilizer as usual
4. After the sterilization process has finished, remove the package and analyze the indicator results. The indicator must turn to the reference color for when it was exposed to Hydrogen Peroxide.
5. If a non-uniform or no color change is noticed, it should be reported immediately to the supervisor.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity. Do not put direct sunlight. Do not re-use. Do not store close to sterilizing agents. Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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