CLEANING INDICATOR HOLDER

YIKAMA İNDİKATÖRÜ APARATI

Product Code/Ürün Kodu AX-WIH01

Size/Ölçü: 32mmx72.5mm

100 cycle / kullanım 1 pcs / adet

compatible with ISO 11140-1:2017

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Manufacturer / Üretici

Kordon Tıp Sağlık Arac ve Gereçleri Ltd.
10006/1 No:43 AOSB Cigli IZMIR TURKEY
Tel:+90 232 348 60 60
Fax: +90 232 348 60 90
www.kordontip.com - info@kordontip.com

CLEANING INDICATOR HOLDER

for washers

Product description

Axis AX-WIH01 Cleaning Efficiency Indicator Holder is designed to mimic the cleaning challenges posed by a large process load during the washing cycle and can be fixed on the tray, which allows reproducible location of the cleaning indicator in the washer-disinfector

When a indicator is placed within the holder, half of the test soil is obscured, thus simulating the joint of a surgical instrument, with water stream hitting it indirectly

The other half of the test soil is covered by a mesh, thus posing a difficulty for water stream to reach it, mimicking shadowed or occluded surfaces

Directions for use

1. Take a **Axis AXCP120 Cleaning Efficiency Indicator** and placed it on the **AXIS AX-WIH01 Holder**. Holder has been designed to mimics the real challenge of thoroughly cleaning hard-to-reach areas for the washing disinfecting process.

2. Place the **AXIS AX-WIH01 Holder** along with **Axis AXCP120 Cleaning Efficiency Indicator** inside and other material packages into the washer. Holder has hooks to ease placement on the tray.

3. Run the washer as usual.

4. Once the cleaning cycle is over, remove the holder with an indicator. Washing efficacy is influenced by detergent, water temperature, time and washing penetration. A good cleaning result is achieved by an appropriate combination of these variables.

5. The Axis recommended of maximum repetition of this process is 100 cycle with the same Holder

Important

Do not touch the indicator ink with your hands. **AXIS AX-WIH01 Holder** is reusable up to 100 cycles.

Disposal

Discard after use according to your country's healthcare and safety regulations.

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