
Wash Indicator Yıkama İndikatörü

@x **AXIS**®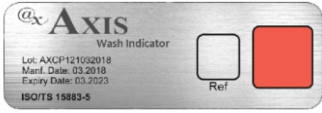

Holder: AX-WIH02

Product Code/ Ürün Kodu: AXCP123

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Wash Indicator

for monitoring efficiency of washing device

Product description

AXIS Washing indicators consist of temperature stable support containing a mixture of specially combined natural colored components (proteins, carbohydrates, fatty acids, and dyes). In all cases, indicator colored test soil is non-toxic, dissolves with most of the cleaning agent and is swept along with the washing fluid. AXIS Washing Indicator cleaning indicators formula allows detection of all the factors that affect the outcome of the washing process. AXIS Washing indicators can be used for routine control of cleaning processes and allow monitoring of washing effectiveness in automatic machines including ultrasonic washing machines with different washing configurations.

Instructions for use

- 1.Choose the AXIS indicator according to the desired procedure
- 2.Place the indicator on AXIS AX-WIH02 holder
- 3.Place the system on the tray along with the material. The holder system can also be used within an empty washer (without material).
- 4.Once the cleaning cycle is over, remove the indicator.
- 5.Analyze AXIS Washer indicators using and record results. If unfavorable results are obtained, perform the appropriate corrective actions and repeat the washing process.

Use at least one AXIS Washing indicator for each cleaning cycle.

Precautions

Do not reuse AXIS Washing indicators. Do not touch the indicator ink with your hands.

Storage

AXIS Washing indicators should be stored at temperatures between 10-30 °C, protected from light, RH 30-80 %, within its original packaging.
NOTE: in case of remaining traces of indicator colored spot after the washing process, do not store the used indicator under direct light since this may deteriorate its coloring.

Unprocessed

Failure-any color rather than reference

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