Ultrasonic Cleaning Test


Ultrasonik Temizleme Testi

@x **AXIS**[®]

Product Code/ Ürün Kodu
AXCP122

24 test / box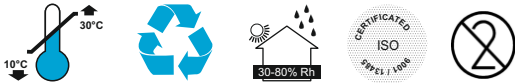

Manufacturer / Üretici

Kordon Tıp Sağlık Arac ve Gereçleri Ltd.
10006/1 No:43 AOSB Cıgılı İZMİR TURKEY
Tel:+90 232 348 60 60
Fax: +90 232 348 60 90
www.kordontip.com - info@kordontip.com

Ultrasonic Cleaning Test

For Ultrasonic Washer

AXCP122

Product description

AXIS AXCP122 indicator was designed to monitor the cavitation process. It allows testing the operation of ultrasonic washing machines with different washing configurations by measuring the generated cavitation energy. AXCP122 indicator consists of a clear vial, with a reactive green-colored solution and glass beads immersed in it. When the cavitation energy is adequate, vibration of the glass pearls triggers a color change in the solution, from green to yellow, through a range of green color intermediates. *Axis Ultrasonic Cleaning Test* is designed for routine control and monitoring ultrasonic cavitation effectiveness of ultrasonic washing machines operating at frequencies equal or greater than 35 kHz, at temperatures ranging from 18°C to 70°C.

Direction for use

- 1-Prepare the tank with cleaning solution following both the manufacturer's instructions of the ultrasonic washing machine and the manufacturer's instructions of the detergent used.
- 2-Degassify the tank according to indications of use of the detergent manufacturer.
- 3-Adjust the temperature according to indications of use of the detergent manufacturer.
- 4-Take the necessary number of indicators according to the Test you want to perform (Periodic Functional Test or Routine Test) and the size of the washer tank.
- 5-Place them in the basket without load according to the distribution of the test to be performed.
- 6-Place the basket in the tank and run the desired ultrasonic cleaning program.
- 7- Completed the program, remove the indicators from the machine.
- 8- Analyze the results according to the table and record the data obtained.

Precautions

Do not re-use the AXIS AXCP122 indicator.
Do not open the indicator before its use, since this will cause its deterioration.
Do not use the AXCP 122 indicator for monitoring normal washing using non-ultrasonic cleaners. At 24 hours, the color of the AXCP 122 indicator might vary. It is recommended to read and register the results when performing the test.

Storage

AXIS AXCP122 indicator should be stored at temperatures between 10-30 °C, specially protected from light and moisture, within its original packaging.

Disposal

Discard the indicators after use according to your country's healthcare and safety regulations.

Interpretation of results

The color change from green to yellow is an accepted result and this indicates the presence of cavitation energy suitable for washing. The time in which this color change occurs is a measure of the strength of cavitations energy. Green color result indicates a failure to reach the cavitation energy necessary for a good cleaning.

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