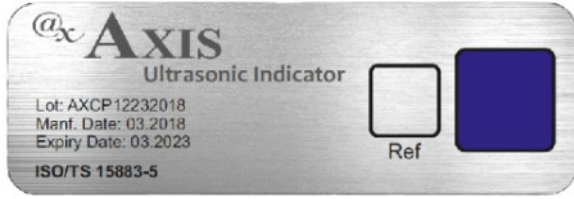
Ultrasonic Washer Indicator

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@x **AXIS**®

Product Code/Ürün Kodu: AXCP123

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Ultrasonic Washer Indicator

for monitoring efficiency of ultrasonic washer device

Product description

AXIS Ultrasonic Washing indicator consist of temperature stable support containing a mixture of specially combined natural colored components (proteins, carbohydrates, fatty acids, and dyes). In all cases, indicator colored test soil is non-toxic, dissolves with most of the cleaning agent and is swept along with the washing fluid. AXIS Washing Indicator cleaning indicators formula allows detection of all the factors that affect the outcome of the washing process. AXIS Washing indicators can be used for routine control of cleaning processes and allow monitoring of washing effectiveness in automatic machines including ultrasonic washing machines with different washing configurations.

Instructions for use

- 2.Place the indicator on AXIS AX-WIH01 or AX-WIH02 holder
 - 3.Place the system with the material. The holder system can also be used within an empty washer (without material).
 - 4.Once the cleaning cycle is over, remove the indicator.
 - 5.Analyze AXIS Ultrasonic Washer indicators using and record results. If unfavorable results are obtained, perform the appropriate corrective actions and repeat the washing process. The indicator ink must be completely cleared for successful result.
- Use at least one AXIS Ultrasonic Washing indicator for each cleaning cycle.

Precautions

Do not reuse AXIS Ultrasonic Washing indicators. Do not touch the indicator ink with your hands.

Storage

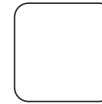
AXIS Washing indicators should be stored at temperatures between 10-30 °C, protected from light, RH 30-80 %, within its original packaging.
NOTE: in case of remaining traces of indicator colored spot after the washing process, do not store the used indicator under direct light since this may deteriorate its coloring.

Unprocessed

Failure-any color rather than reference

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