WASHING ACTIVITY INDICATOR YIKAMA ETKİNLİK İNDİKATÖRÜ

Product Code / Ürün Kodu: AXCP125

@x **AXIS**®

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Washing Activity Indicator

for verification of the cleaning efficacy of washer-disinfectors

Product Description

Axis Cleaning Efficiency Indicator enables you to verify the cleaning efficacy of washer-disinfectors during routine operation check. The routine use of this test will help ensure that your washer is performing at a consistent level, enhancing the routine visual inspection of instruments.

If unfavorable results do occur, the users must check the quantity of detergent, cleaning time, temperature as well as general condition of the washer.

Ready-to-use (single use product)
The test provides efficient and reliable results
Transparent cover to enable visual check

Instructions of use

1. place the envelope with the washing indicator inside the equipment, together with the load to be processed. WARNING! The indicator must be completely submerged in the washing solution to work properly.
2. Process the load according to the established procedures.
3. After completing the process, analyze the results. The indicator must change to the reference color for considering that the desired disinfection conditions were met. The color in the indicator must be homogenously cleaned and removed in the indicator surface

Storage and shelf life

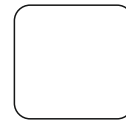
- Keep this indicator in dry places, protected from light and at temperatures between 10-30 °C and 30-80 % relative humidity. Do not wet. Do not storage close to sterilizing or disinfection agents.
- Indicators have an expiration date of 5 years from the date of manufacture when stored in the recommended conditions. Both expiration date and batch of the product are specified on the box label and on each unit of product. Do not use indicators after their expiration date.
- After use, the indicator will not change in 1 year, if stored at the temperature and relative humidity previously indicated.

Unprocessed

Failure- all colors except
reference

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