
Container Label Konteyner Etiketi

@x
AXIS

Product Code/Ürün Kodu
AXCP200

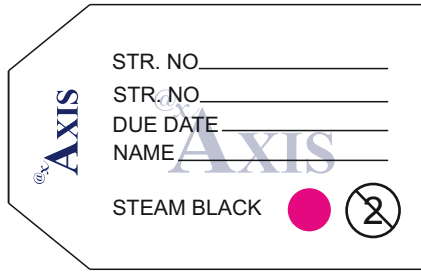100 pcs/adet

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Container Label

for steam sterilization processes

Product Description

Container seals for Steam sterilization processes.
Conditions: 10 minutes 121 °C ; 2 minutes 134 °C.
Initial color: pink
Final color: siyah

Container labels are composed of a identification card with an chemical indicator. The area provides the space for identification of sterilizer no, sterilization date, due date an name. When label exposed to steam the pink indicator turns black.

Application

The product is used to identify sterilized materials in the sterilizer. It is placed inside the load according to the regulations of the institution or hospital. The pink indicator on the label turns black when exposed to steam. It is placed on the container in order to ensure the exposure and sterilization safety of the container after sterilization.

Storage

Indicators should be stored in dry places, protect from light at temperatures between 15 - 30 °C, 35 - 60 % relative humidity.
Do not wet.
Do not storage close to sterilizing agents.
Do not use indicators after their expiration date.
Do not reuse.

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