Container Seal with Chemical Indicator Kimyasal İndikatörlü Konteyner Kilidi

@x **AXIS**®

Product Code/ Ürün Kodu:
AXCP220

250 pcs/adet

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Container Seal

for monitoring Steam sterilization processes

Product description

Axis AXCP220 Container Seal with Chemical Indicators are designed to react to Steam sterilization processes. Pink indicator turns to reference color of Black when the load exposed to Steam sterilant. The plastic security lock is locked using disposable plastic security locks, these locks can only be opened by breaking. The container is locked before the sterilization process and acts as a security to prevent unauthorized persons from opening it after the sterilization process. It is the method that can be used to mark and identify the containers, trays, devices to be sterilized.

Directions for use

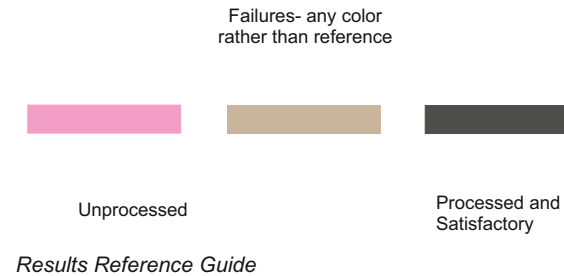
1. Place the Axis AXCP220 Container Seal on each container that will be sterilized by Steam.
2. Plastic safety lock is disposable, used by locking containers to secure the lid cover after sterilization until surgery, which can only be opened by breaking.
3. Run the sterilizer as usual with the container inside.
4. The container is locked before the sterilization process and acts as security to prevent unauthorized persons from opening it after the sterilization process.
5. For to check exposed and unexposed containers identification compare the results from the given reference color chart. If the color change to reference color of black means that the container has been exposed to Steam.
6. After use, the seal can only be opened by breaking.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.
Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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