HELIX Process Challenge Device HELIX Proses Zorluk Cihazı (PCD)


@x **AXIS**®

Product Code/Ürün Kodu: AXHELIX-1.5

Manufacturer / Üretici

Kordon Tıp Sağlık Arac ve Gereçleri Ltd.
10006/1 No:43 AOSB Cıglı IZMIR TURKEY
Tel:+90 232 348 60 60
Fax: +90 232 348 60 90
www.kordontip.com - info@kordontip.com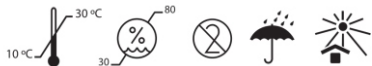

HELIX Process Challenge Device

Product Description

- AXIS Helix-1.5 devices have been designed to evaluate the penetration capability of the sterilizing agent (Steam, Ethylene Oxide, Formaldehyde and Hydrogen Peroxide). For a successful sterilization, the sterilizing agent must contact all external and internal surfaces of the item to be sterilized. To achieve this, air and other non-condensable gases (NCG) must be evacuated from the sterilizer chamber prior to the sterilant injection. AXIS Helix-1.5 devices simulate hollow instruments of different levels of complexity allowing testing of the penetration capability of the sterilizing agent. AXIS Helix-1.5 devices consists of a metal or plastic capsule of great resistance and durability, which is able to hold a chemical indicator designed for this system. The capsule has a screw cap which allows the introduction of the indicator and is connected to a high resistance plastic tube of variable length. The plastic tube is helically arranged and it must not be released when using the product. A connector coated and reinforced by an external connector, holds the tube and capsule joint. AXIS Helix-1.5 devices with the appropriate indicator must be processed together within the load. After the sterilization cycle, the result must be observed. Homogeneous color change of the chemical indicator ink means the sterilization process was successful and the load can be securely released.

Handling information

- 1- Choose an appropriate chemical indicator for the sterilization cycle to evaluate.
- 2- Fold the indicator strip in the middle through the punched line, the reactive ink must be facing the inside. Avoid folding the printed zone.
- 3- Place the chemical in the screw cap pin (metal capsule) or in the screw cap cavity (plastic capsule). Put the folded end first.
- IMPORTANT: Avoid contact between chemical indicator and capsule inner surface, since the results can be altered.
- 4- Put the capsule cap by applying moderate pressure.
- 5- Place the system into autoclave.
- 6- Run the desired sterilization cycle.
- 7- Once the sterilization process is finished, remove the AXIS Helix-1.5 from the sterilization chamber.

- CAUTION: For Steam processes, gloves should be worn. Also, allow 15 min for the load to cool down in order to avoid burns.
- For EO, Formaldehyde or Hydrogen Peroxide processes, gloves, glasses and mask should be worn to avoid contact with the agent remains. For EO, a 24-hour ventilation period or a time period suggested by the manufacturer, should be taken into consideration.
- 8- Remove the indicator and evaluate the result.
- 9- Record results

Chemical Indicator

- Choose the appropriate chemical indicator according to the sterilization process to be evaluated.

AXHLX4(Steam).....	4 bars
AXHLX6(Steam).....	6 bars
AXHLX10 (EO).....	4 bars
AXHLX30(Formaldehyde).....	4 bars
AXHLX40 (Plasma).....	4 bars

Additional technical information


- In each cycle, the user should check that the connector and the plastic tube are in perfect conditions. NOTE: In case the connector is damaged and replacement is needed, contact a sales representative and request a replacement. In case the tube is blocked or has leakages, the complete device should be replaced.

Storage

- AXIS Helix-1.5 device and indicators should be stored in dry places, at temperatures between 10-30 °C, 30-80% RH, protected from light. Do not store close to sterilizing agents. Do not use indicators after their expiration date.

Caution

- Kordon Tıp Ltd. does not guarantee proper functioning of the AXIS Helix-1.5 device when used with indicators of other manufacturers. AXIS Helix-1.5 device results are only valid for the cycle run. Do not use them to draw conclusions about previous or future cycles. AXIS does not guarantee proper functioning of the AXIS Kit Helix-1.5 when used for sterilization processes other than the process for which it has been designed.

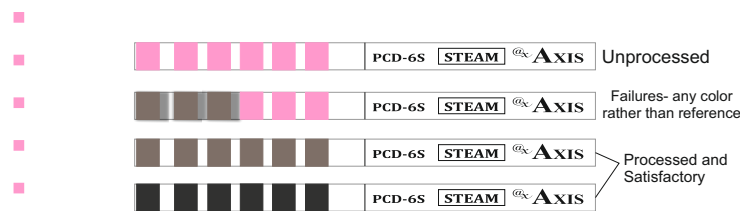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