@x AXIS[®] TYPE-1 AXS10

Quality Certificate/ Kalite Sertifikası

48 hour
saat

Lot : Population : CFU/KOB
Bacillus atrophaeus (ATCC 9372) Popülasyon

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D-Value : min./dk.
D-Değeri
(54°C, 60% RH, 600mg EO / liter)

Survival Time : min./dk.
Hayatta Kalma Süresi
(Survival time = not less than (log₁₀ labeled population - 2) x labeled D-value)

Kill Time : min./dk.
Ölüm Zamanı
(Kill time = not more than (log₁₀ labeled population + 4) x labeled D-value)

EO

D-Value : min./dk.
D-Değeri
(160°C)

Survival Time : min./dk.
Hayatta Kalma Süresi
(Survival time = not less than (log₁₀ labeled population - 2) x labeled D-value)

Kill Time : min./dk.
Ölüm Zamanı
(Kill time = not more than (log₁₀ labeled population + 4) x labeled D-value)

Valor Z : °C
Z-value

DRY

100 pcs / adet

Manufacturer / Üretici
KORDON TIP SAĞLIK ARAC GEREÇLERİ
MUH.PROJE.İTH.SAN.VE TIC.LTD.STİ
10006/1 Sok. No:43 35620 A.O.S.B CİĞLİ İZMİR TURKEY
+90 232 348 60 60 / +90 232 348 60 90
www.kordontip.com - info@kordontip.com

AXIS IS A BRAND OF KORDON TIP LTD.

Spore Strip

for monitoring Eo & Dry Heat processes

Product Description

Each Axis AXS10 Spore Strips consists of a paper strip soaked with *Bacillus atrophaeus* ATCC 9372 spore population of 106.

48 hours of incubation at 37± 2°C.

Content

The Axis AXS10 Spore Strips designed for monitoring Ethylene Oxide or Dry Heat sterilization processes.

Instructions for use

1. Identify the Axis AXS10 Spore Strips by writing details.
2. Pack it in an appropriate package along with the material to be sterilized, according to recommended sterilization practices. Place the envelope in those areas considered more inaccessible for sterilizing agent. A typical problematic area is the load of the center.
3. Run the sterilizer as usual.
4. After the sterilization process has finished, wear safety glasses and gloves than remove the Axis AXS10 Spore Strip from the sterilized package for processing and incubation. For Ethylene Oxide sterilization, you can aerate the package containing the biological indicator with the rest of the load or remove the biological indicator from the sterilization package for incubation in any appropriate growth medium before material aeration.

IMPORTANT!

Use latex gloves when transferring spore strips from the envelope to growth medium tube. Avoid spore strip contact with any kind of surface including the outer part of the growth medium tube and of Axis AXS10 Spore Strip envelope.

Use a non-sterilized spore strip as a positive control at least once per day, when a sterilization cycle is run. Both, processed spore strip and the strip used as positive control must belong to the same batch. Positive control guarantees that the incubation was carried out under appropriate conditions.

Incubate processed spore strips along with the strip used as positive control during a maximum of 48 hours at 37±2°C. It is advisable to make observations every 10 hours. The color change of the growth medium to yellow means a failure on sterilization process. If after 48 hours no colour change is visible on growth medium, the result is negative (i.e. the sterilization process was successful).

Storage Conditions

Keep goods in their original packages.
Temperature between 10-30°C and 30-80 % relative humidity.
Do not freeze. The strips should not be stored near sterilants or other chemicals.
Axis AXS10 Spore Strip has a 24 month of shelf life when stored at recommended conditions.
Do not use strips after their expiration date.

Disposal

Dispose used spore strips per your facility procedure. The positive strips can be autoclaved at 121°C for at least 20 minutes, or at 132°C for 15 minutes in a gravity displacement steam sterilizer, or at 134°C for 10 minutes in a vacuum-assisted steam sterilizer.

Warnings:

! Do not use Axis AXS10 Spore Strip for monitoring Steam sterilization cycles, radiation or other sterilization processes differs from EO and/or Dry Heat sterilization.
Do not reuse spore strips.

Axis AXS10 Spore Strip is compliant with ISO 11138-1:2017 and ISO 11138-2:2017 (ISO 11138-1:2017 and ISO 11138-2:2017).

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