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MULTIPARAMETER INDICATOR

for steam sterilization processes

Product Description

AXIS Multiparameter Process Indicators are single-use chemical indicators that consist of one-side laminated punched paper double strips printed with indicator ink. These indicators have been designed to monitor Steam sterilization processes within loads, ensuring an adequate exposure to the sterilizing agent during the sterilization process and allowing to distinguish between processed and unprocessed items. Innovative highly sensitive pink ink was developed to turn to dark brown/black when the indicator is exposed to Steam considering the critical process variables which provides additional quality assurance.

Instructions for Use

1. Place a AXIS Multiparameter Process Indicator within each package, bag or tray that will be sterilized by Steam. In rigid containers, place the indicator in every corner, or at least two diagonally opposite corners, to ensure that the sterilization agent has reached the most inaccessible areas.
2. Perform the sterilization cycle.
3. After the sterilization process has finished, remove the indicators from the sterilizer and analyze the results. For all the chemical indicators exposed to Steam, the ink must turn to the dark brown/black reference color when the sterilization parameters are reached. For reference color, please refer to Result Reference

Storage

Indicators should be stored in a dry place, protected from light at temperatures between 10-30 °C, 30-80 % relative humidity. Do not wet. Do not store near sterilizing agents.

Shelf Life

Chemical Indicators have an expiration date of 5 years from the date of manufacture when stored at recommended storage conditions. Both the expiration date and batch number of the product are specified on the package label and in each unit of the product. Do not use indicators after their expiration date.

Precautions

Do not expose AXIS Multiparameter Process Indicators to Ethylene Oxide, Dry Heat or any sterilization process other than Steam. Do not reuse.

Disposal

Discard chemical indicators after use along with paper waste, according to your country's healthcare and safety regulations.

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Manufacturer: Kordon Tıp Sağlık Arac ve Gereçleri Ltd.