EMULATING DOUBLE SIDED INDICATOR EMÜLATÖR ÇİFT TARAFLI İNDİKATÖR

@x **AXIS**®

Product Code/Ürün Kodu
AXSDC6
CLASS 6

REFERENCE GUIDE/REFERANS KILAVUZU

UNPROCESSED

FAILURE

PROCESSED AND SUCCESSFUL

Manufacturer / Üretici

Kordon Tıp Sağlık Arac ve Gereçleri Ltd.
10006/1 No:43 AOSB Cigli IZMIR TURKEY
Tel: +90 232 348 60 60
Fax: +90 232 348 60 90
www.kordontip.com - info@kordontip.com

EMULATING DOUBLE SIDED INDICATOR for monitoring steam sterilization processes

Product description

AXIS Emulating Indicator are designed for internal use in steam sterilization processes, for ensuring the achievement of all the critical variables (temperature, time, steam quality). The pink ink was developed to change to black when the process reaches the stated values (SV) of the critical process variables indicated in the emulator.

Instructions of use

- Place an AXIS Emulating Indicator within each package, bag or tray that will be sterilized by Steam.
- In rigid containers place an emulator indicator in every corner, or at least two diagonally or opposite corners, to ensure that the steam has reached the most inaccessible areas.
- Finished the process, analyze the results. The indicator must change to the reference color for considering that emulation conditions were reached.

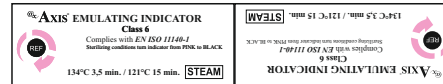
Storage and shelf life

- Indicators should be stored in dry places, protect from light at temperatures between 10-30 °C, 30-80 % relative humidity. Do not wet. Do not store close to sterilizing agents.
- Indicators have an expiration date of 5 years from the date of manufacture when stored in the recommended conditions. Both expiration date and batch of the product are specified on the package label and on each unit of product. Do not use indicators after their expiration date.
- After use, the indicator shall remain unchanged for a period of not less than 6 months when stored at temperature and relative humidity previously indicated.


Disposal

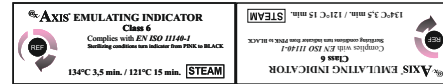Discard along with cardboard waste according to your country's healthcare regulations.

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