HELIX 6 Line Helix Test Kit Set

HELIX 6 Satırlı Helix Test Kit Seti

@x **AXIS**®

TYPE 2

Product Code/ Ürün Kodu
AXSDHLX6

100 pcs / adet

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Helix Test Kit

AXSDHLX6

for monitoring Steam sterilization processes

Product description

Axis AXSDHLX6 Helix Test indicator strips are designed for measuring air removal and steam penetration capability in pre-vacuum steam sterilizers. They provide immediate visual evidence of inadequate air removal or air leak, vacuum pump failures.

Directions for use

1. Take 1 strip from AXSDHLX6 Helix Test Chemical Indicator. Open the cover at the end of the metal part. Fold the indicator strip in the middle through the punched line, the reactive ink must be facing the inside.
2. Place the chemical in the cap cavity (plastic capsule). Put the folded end first.
3. Put apparatus inside the tray. Run the sterilizer as usual.
4. Once the sterilization process is finished, remove the Helix Indicator.
5. Compare the results from the given reference color chart. If the color change to reference color of Brown consider that the established sterilization conditions were met. If one or more bars did not reach the final color, the load cannot be considered sterile.
6. After use, for recording, the indicator will remain its final color for 6 months. It is non-toxic, does not contain lead and does not transfers to materials to be sterilized.

Storage

Store it in a dark place between 15-30°C, 35-60% relative humidity.
Do not put direct sunlight.
Do not re-use.
Do not store close to sterilizing agents.
Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions.

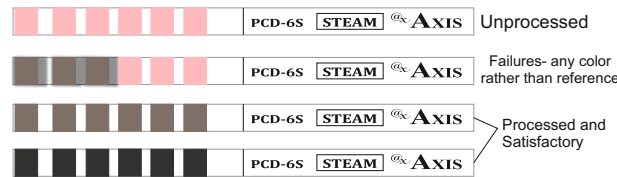
Important

If the indicator does not properly turn to Brown color, these may be the reasons;
• choosing a Helix apparatus of inappropriate length

Caution! The plastic tube is helically arranged and it must not be released when using the product.

Disposal

Discard after use, according to your country's healthcare and safety regulations.

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